


DIR[®]


Stirrups For Exam Chairs

ACCS-C8-OB


INSTRUCTION MANUAL

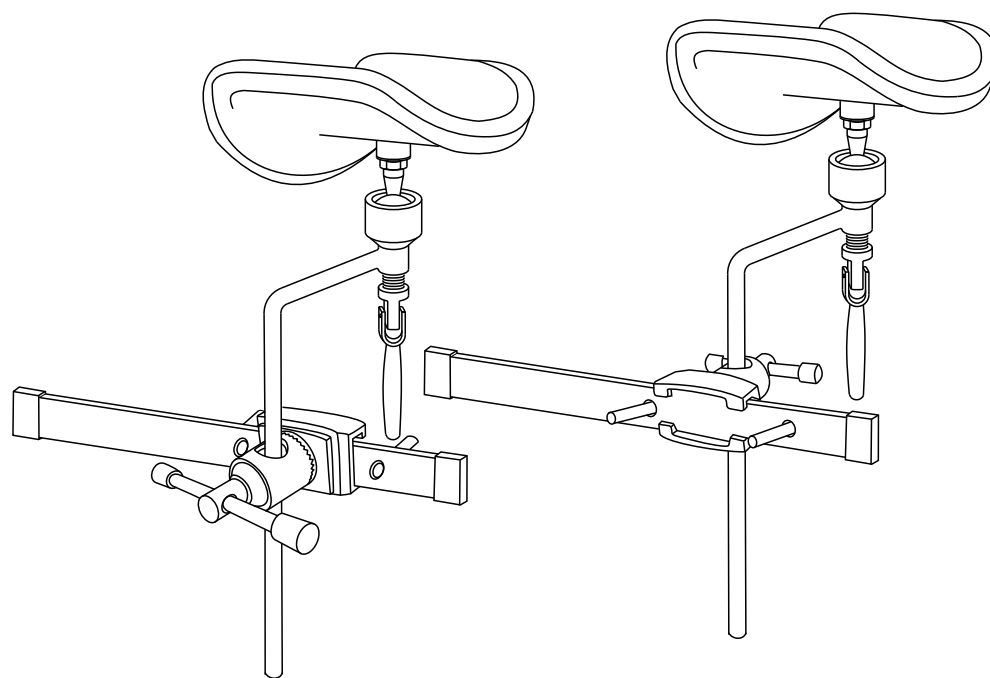


List of Parts & Tools

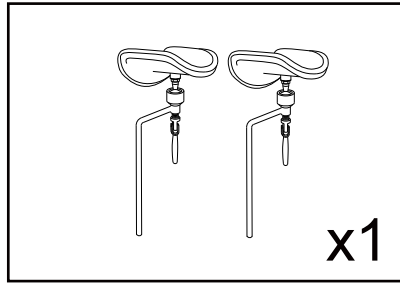


Leg Rest

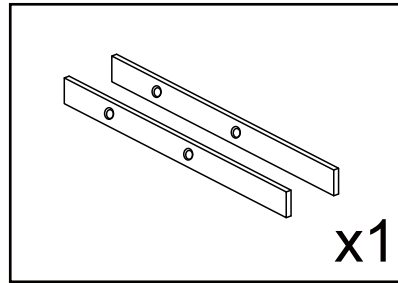


Lead Rail

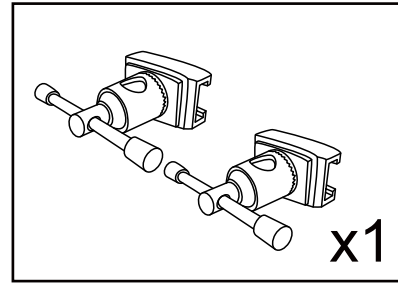


Quick-release Lock

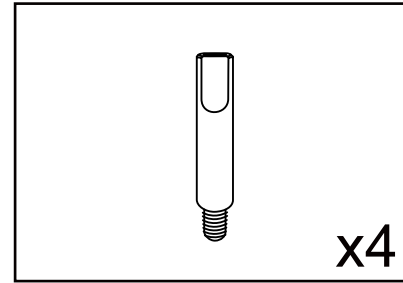


Threaded Rod

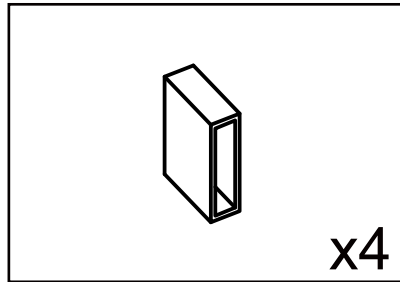


Plastic Sleeve

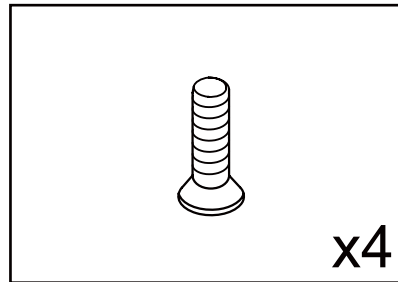


M8x20mm

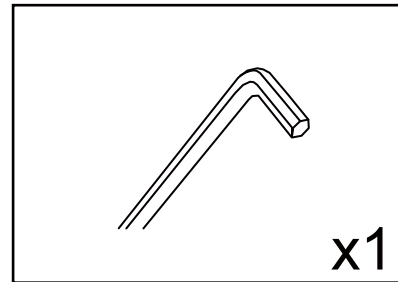

5mm


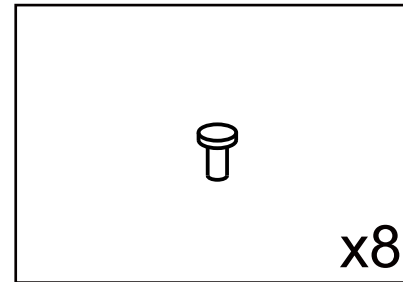


Rubber Plug


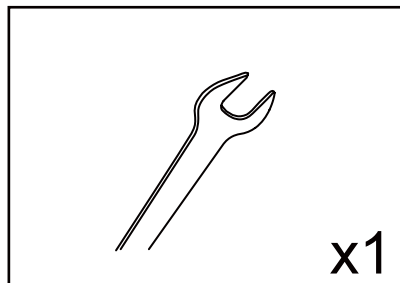


Wrench

Model:8103-3G

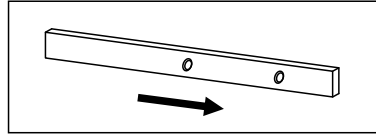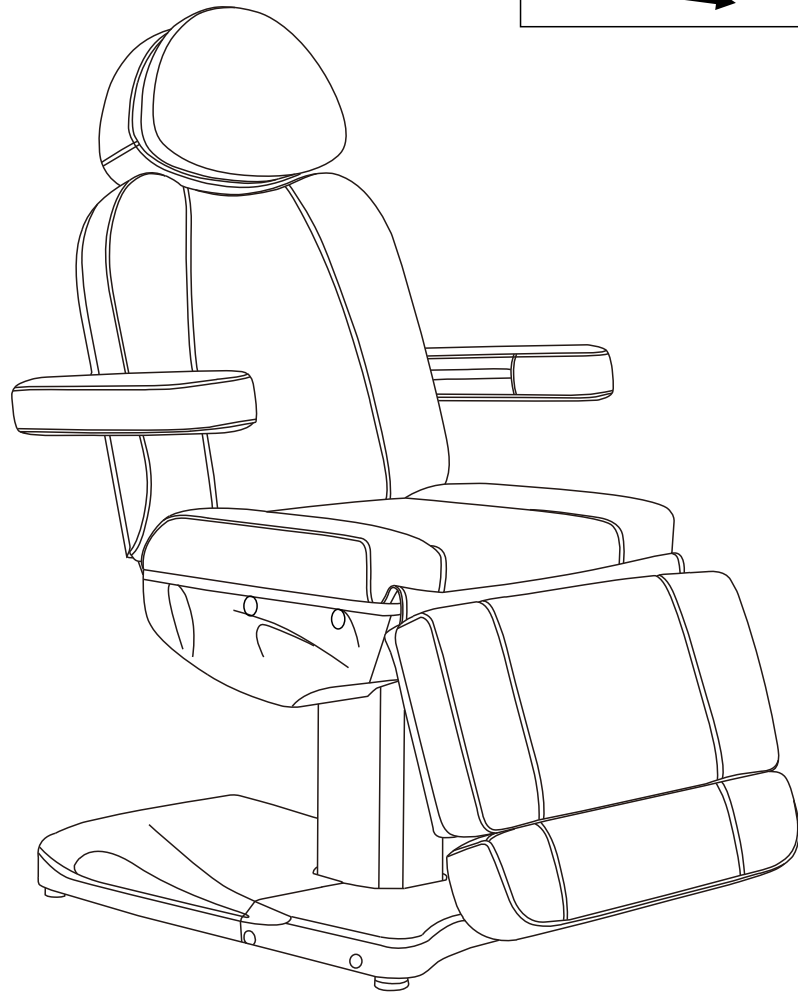

Model:8709-3GAM

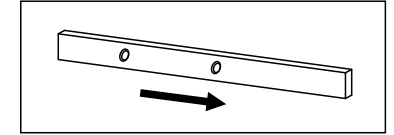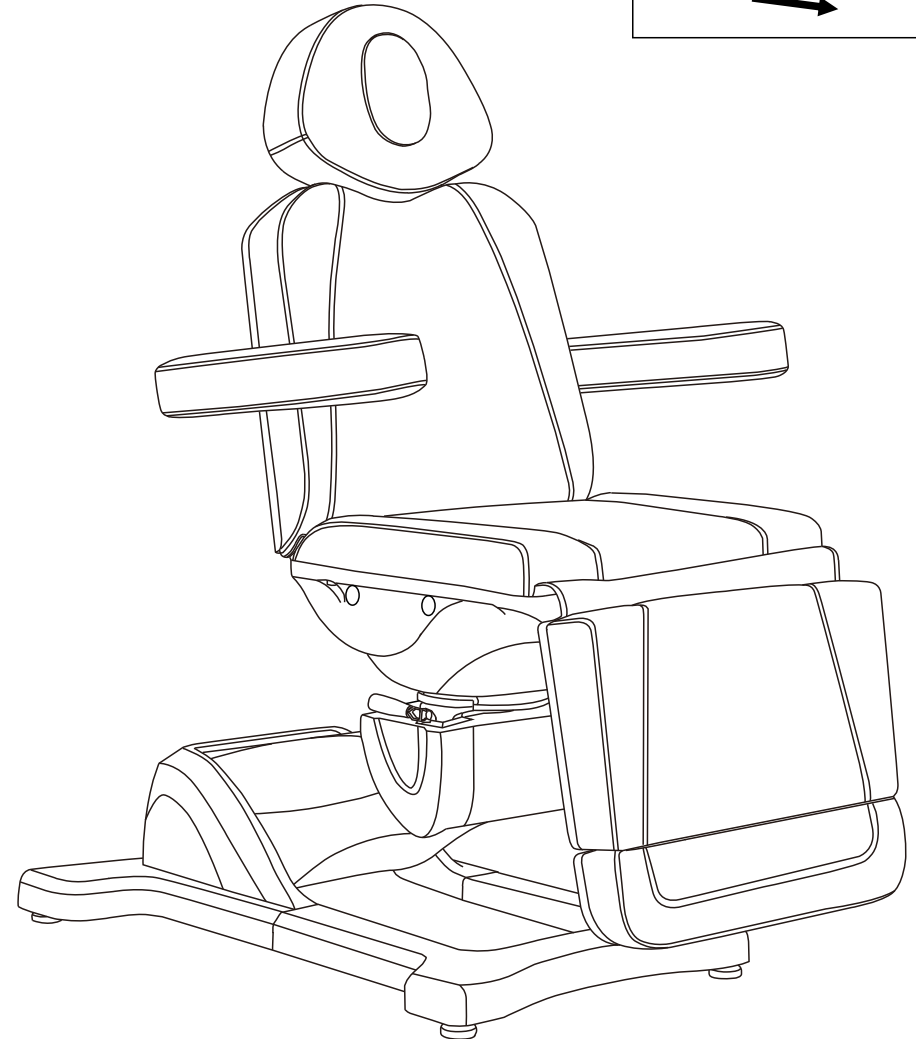


The mounting direction of guide rails varies by bed model. Orientation must be determined with reference to the two screw holes as the central datum point:

Model 8103 Configuration: Front orientation: Short end of rail faces toward leg plate assembly


Model 8709 Configuration: Front orientation: Long end of rail faces toward leg plate assembly

Step 1

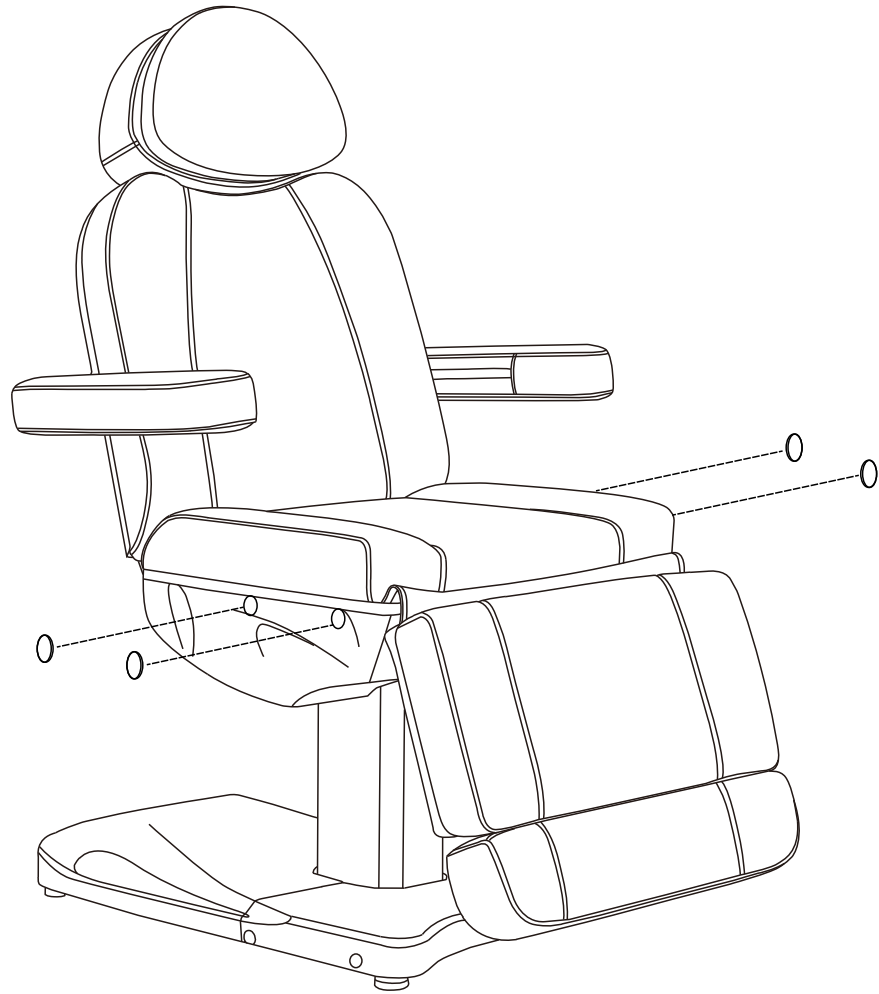




Pry out the rubber plug using a flathead screwdriver.

Step 2

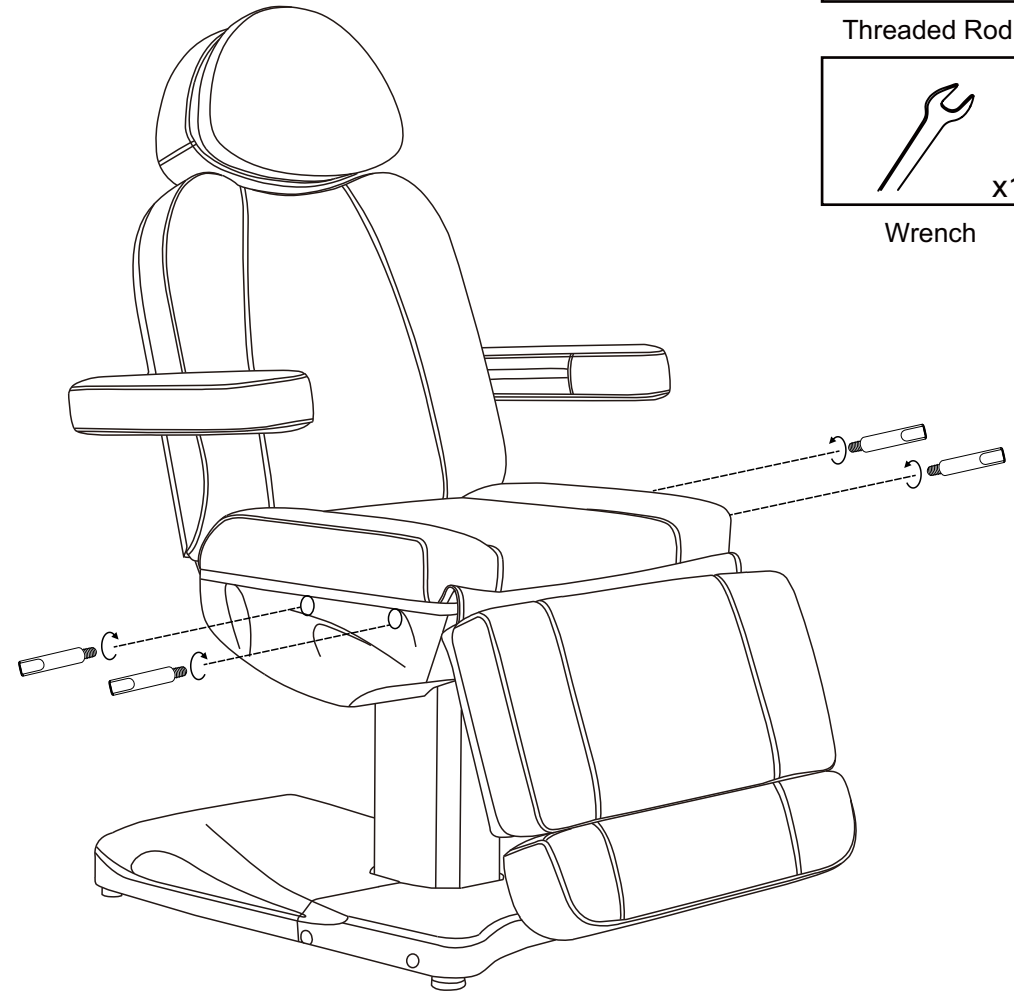


Position the metal screw rod into the mounting hole and turn it clockwise to secure.


Step 3

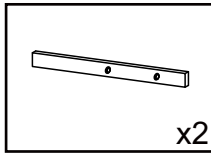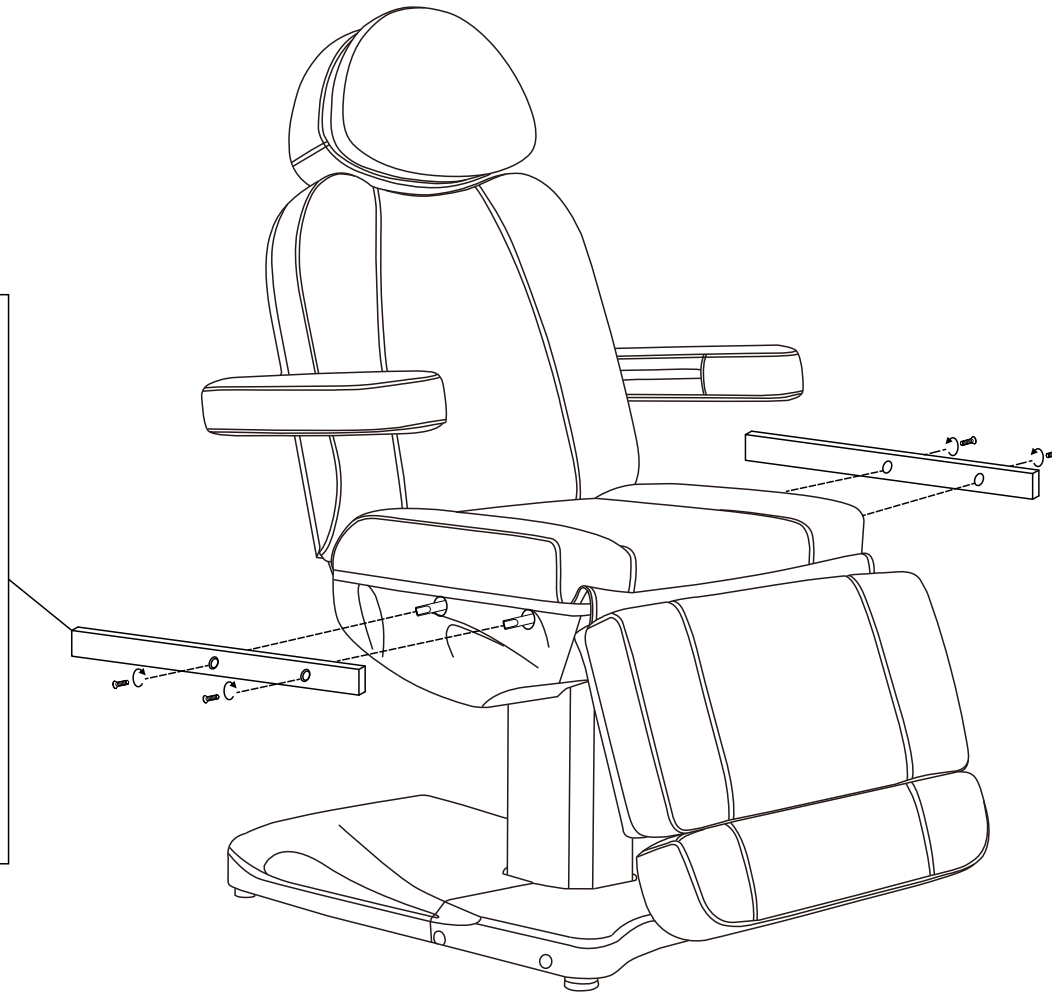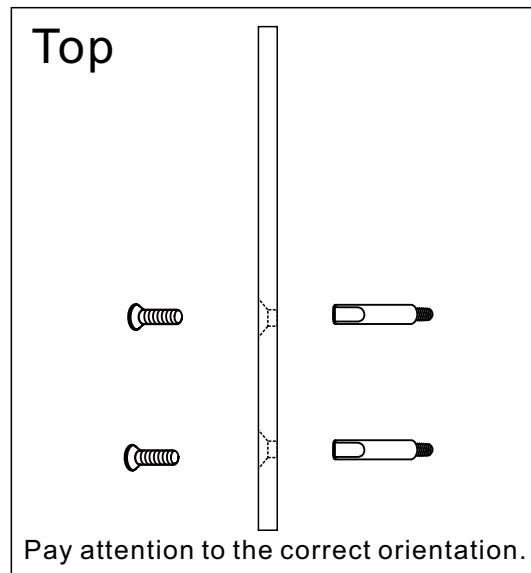

Lead Rail

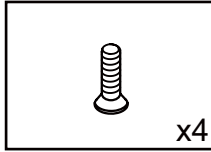

M8x20mm

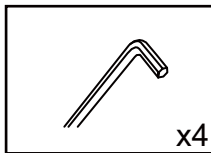


5mm

Position the rail so that its mounting holes align with the metal studs. Insert the screws and tighten them with a hex wrench.

Note: The 8103 rail must be installed with its shorter end toward the leg panel, while the 8709 rail requires the longer end facing the leg panel.

Important: The smooth (flat) side of the rail should contact the threaded rods,

Step 4

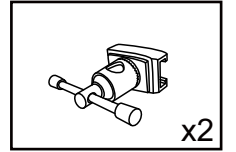


Quick-release Lock




Slide the quick-release lock onto the rail.

Step 5

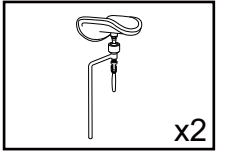


Leg Rest

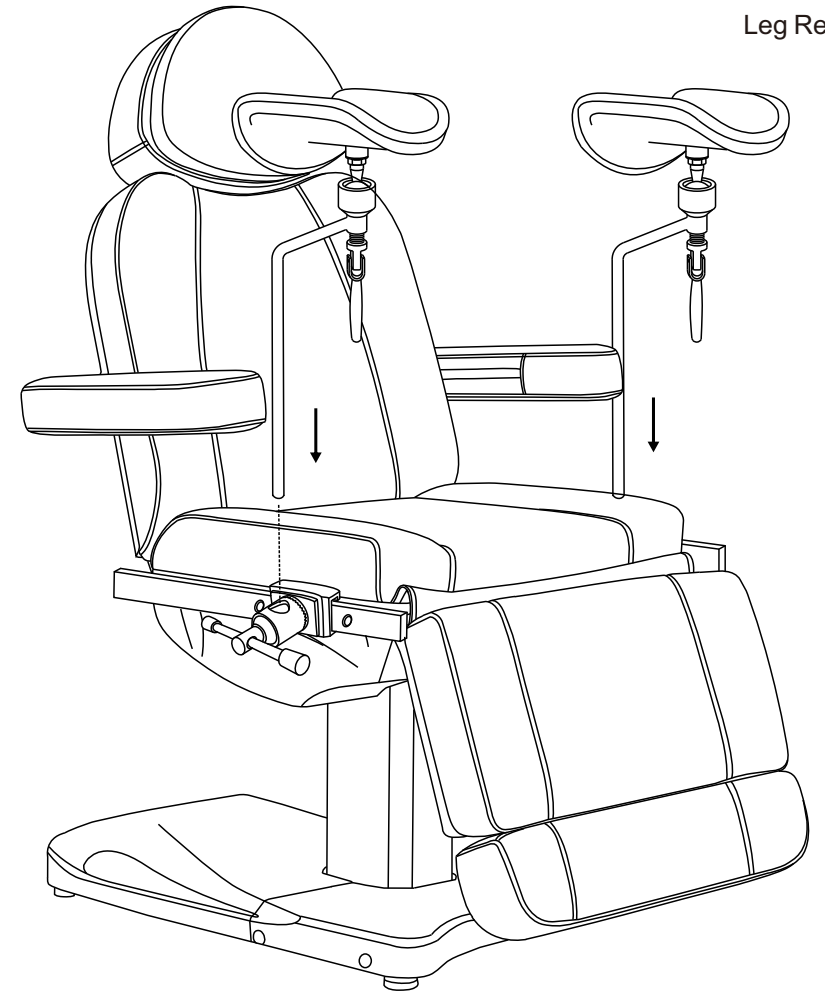


Insert the iron rod of the leg rest into the hole of the quick-release lock.

Step 6

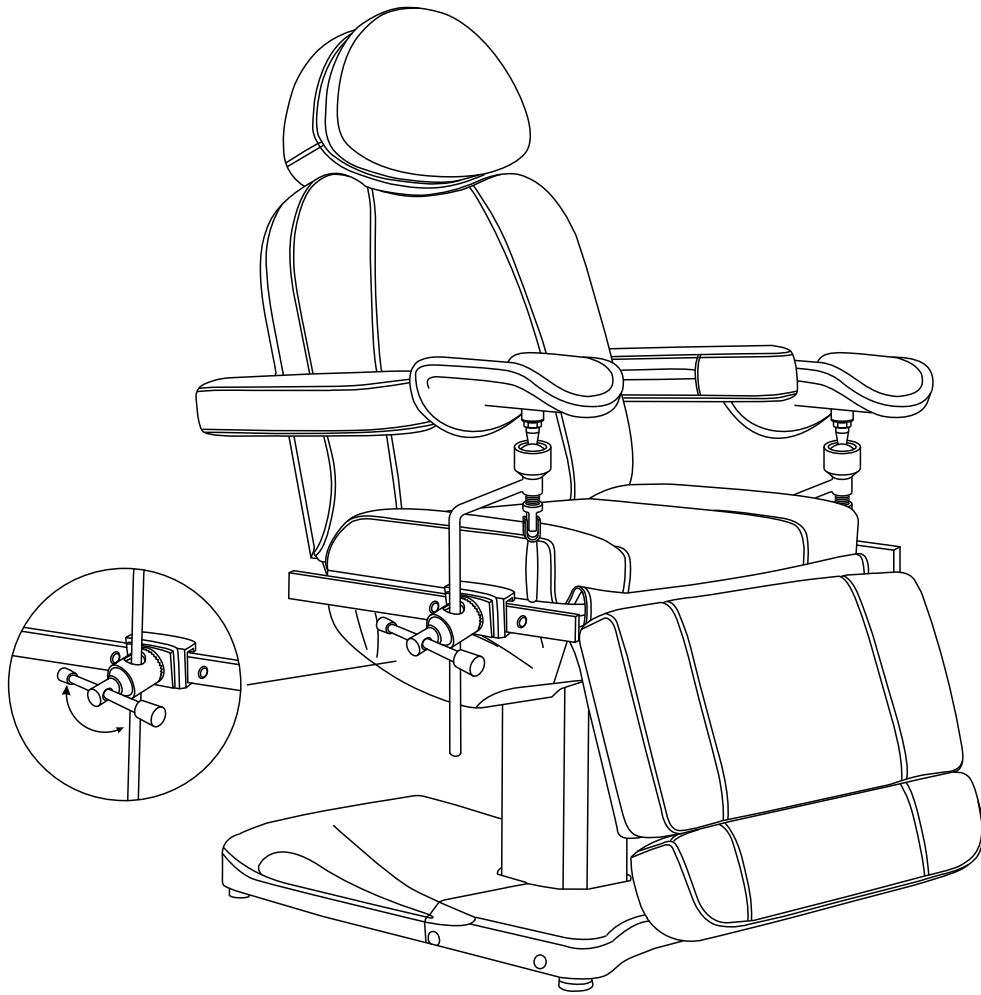




After positioning the leg rest in the proper place, rotate the round rod on the head of the quick-release lock clockwise to secure it.

Step 7

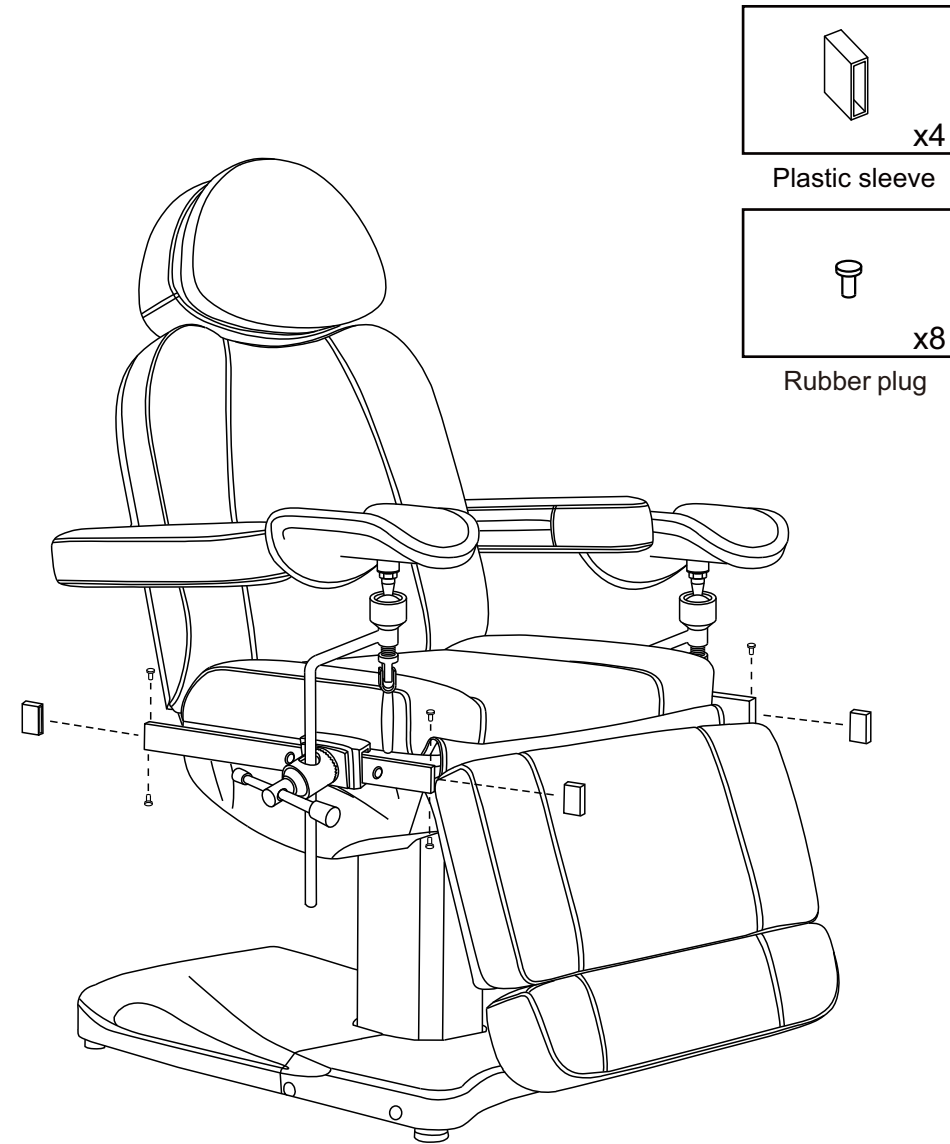


insert the rubber stoppers into the holes at both ends of the guide rails, then slide the plastic sleeves over both ends of the guide rails.

Instructions for use

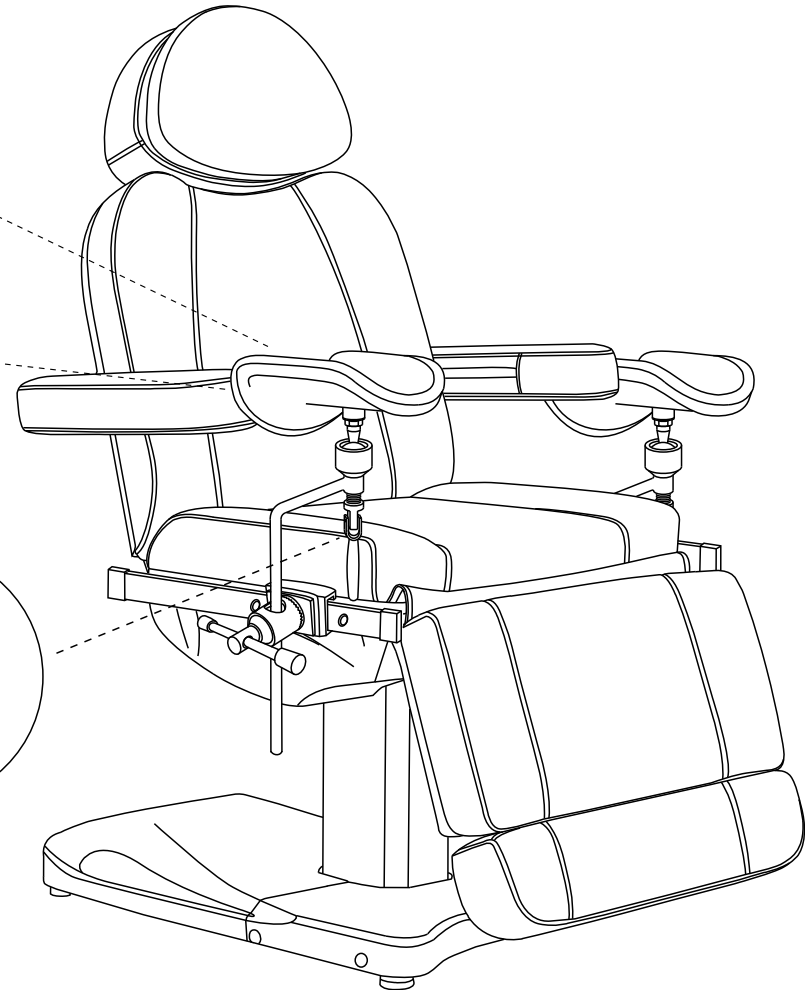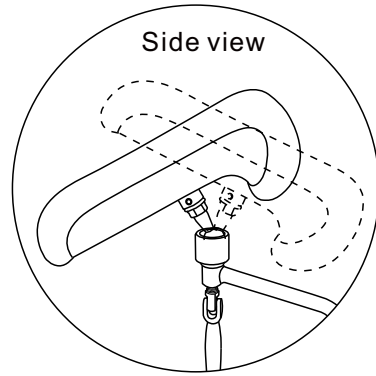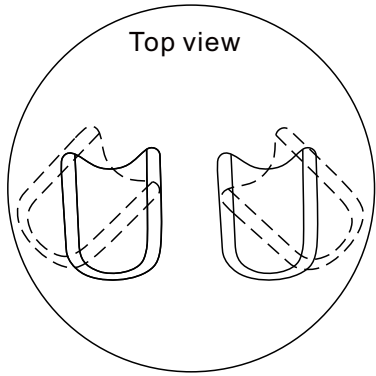


When the leg rest is in the unlocked state, the leg-supporting part can rotate in multiple directions.

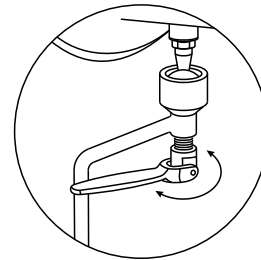

Rotate the lock under the leg rest clockwise to secure the leg rest.

Instructions for use

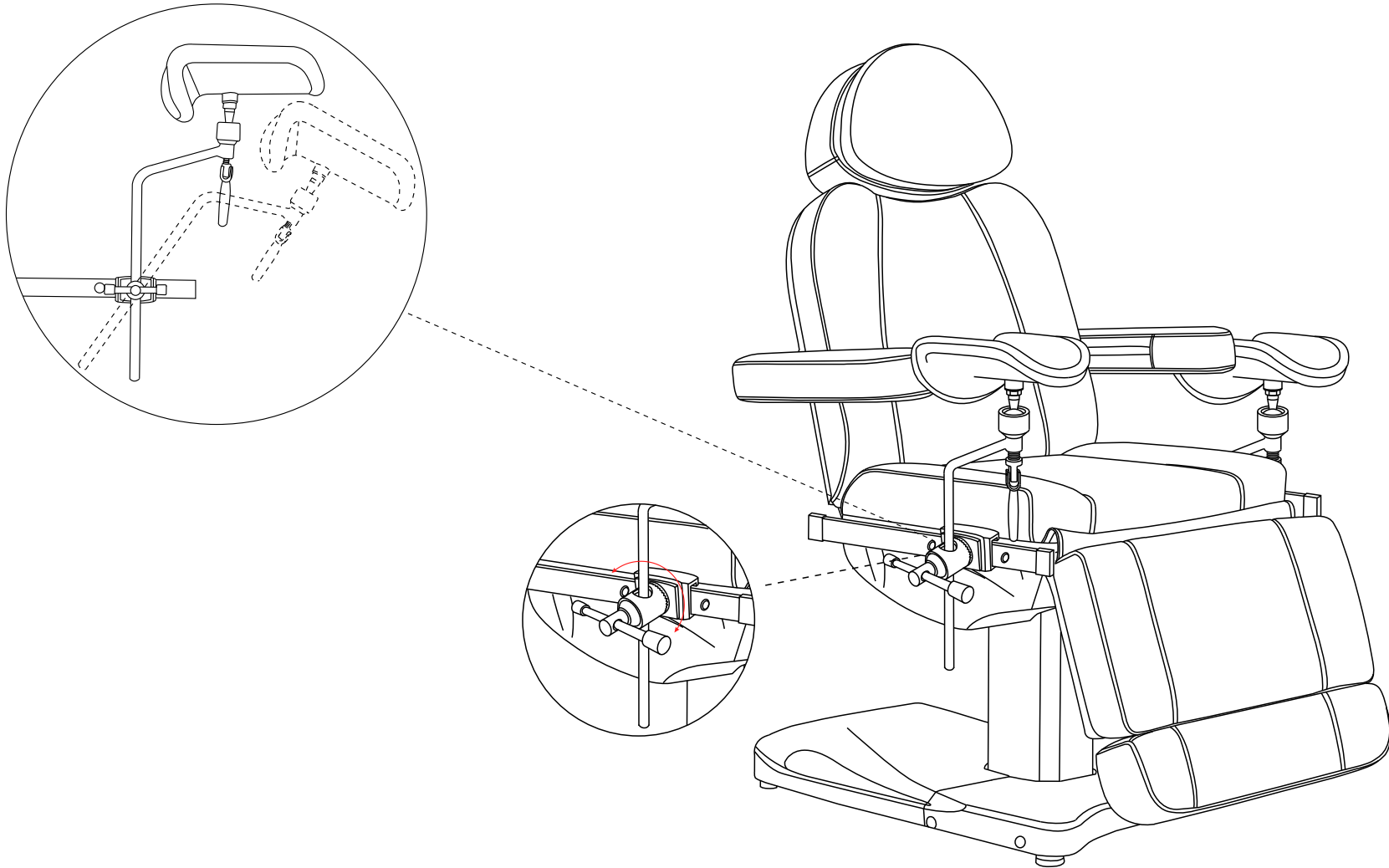


The quick-release lock, when inserted into the leg rest section, is capable of a 360-degree rotation.


Product care




Brush away excess hair

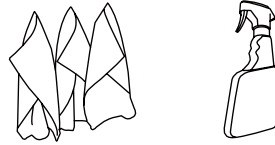

Clean surface with damp cloth or use simple soap & water solution.

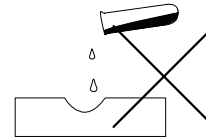

Avoid using corrosive cleaning agents such as bleaches.

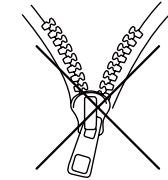

Do not remove upholstery for cleaning

Warranty

The warranty period is one year. DIR warrants to you, the original buyer, that goods shall be free of defects in material and/or workmanship attributable to manufacture for one year from date of receipt of product. Damage resulting from improper handling after merchandise is received, or from exposure to weather or extreme atmospheric conditions is not covered by the warranty. For any other information please contact our support team:

DIR Birmingham

66 Rea Street South
Birmingham
B5 6LB

Email: sales@dirstore.co.uk
Phone: +44 (0) 121 6434777

DIR London

Unit 1 River Road Business Park,
33 River Road, Barking,
Greater London, 1G11 0DA

Email: london@dirstore.co.uk
Phone: +44 (0) 20 8591 9310

DIR New York

575 Underhill Blvd Suite #170
Syosset, NY 11791, USA

Email: sales@dirsalonfurniture.com
Phone: +1 (718) 628 6668
+1 (800) 728 2008
Fax: +1 (855) 628 5168